

Dosing Calculator Quick Start Guide

For HCPs only



Scan to access the calculator

or visit www.calculator.dojolvihcp.com


You can access the website on your desktop, laptop, tablet or smartphone





Dosing Calculator

For HCPs only

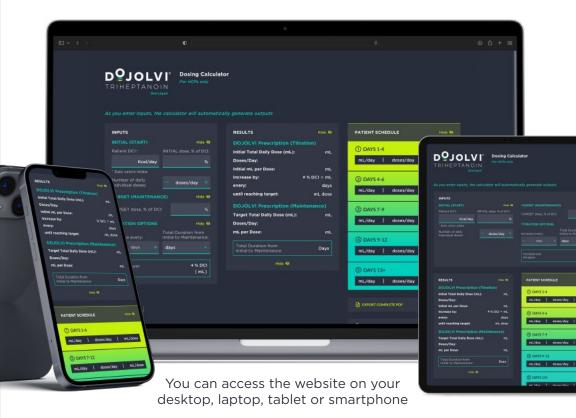

Designed to help you calculate the DOJOLVI daily dosing schedule for your patients.

- Works for all patients ready to start DOJOLVI
- Calculates:
 - initial dose
 target dose
 titration doses
- Exports a PDF:
 - for your files showing all inputs, outputs and a patient schedule
 - for your patient with mL/dose and daily dose schedule for entire titration period

1. Select your patient type



It is best to review this guide while using the calculator

You will be asked to confirm you are a Healthcare Professional before using

- a Patient type selection
- Belevant dosing recommendations provided from the Prescribing Information (PI)
 Full PI available for review

2. Enter the values for dosing information



Show/Hide buttons to open/close sections

- c DCI (daily caloric intake)
- d INITIAL dose, % of DCI
- e Number of doses per day
- TARGET dose, % of DCI
- g Increase every # days
- h Total duration (multiples of the # of days selected in part g)
- i Calculates in real time based on inputs above


3. Review results and patient schedule



Results section will populate in real time as you enter inputs

Patient schedule shows relevant information for your patient

4. Download and share results

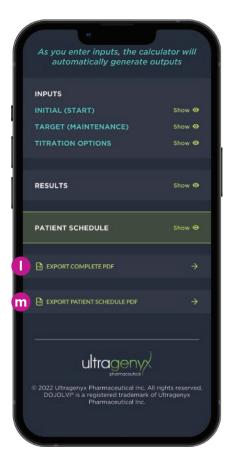

- Complete PDF of all inputs, outputs and the patient schedule for your files
- Patient schedule PDF

 for your patient

 (you can handwrite or type
 notes and a daily dose schedule)





Oral Liquid

If you have questions about this calculator, please reach out to your Ultragenyx representative or visit www.doiolvihcp.com



Access the calculator now

Scan to access the calculator or visit www.calculator.dojolvihcp.com



© 2022 Ultragenyx Pharmaceutical Inc. All rights reserved, DOJOLVI® is a registered trademark of Ultragenyx Pharmaceutical Inc. MRCP-UX007-00413 02/2022

